# FR Urimed® Vision

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# GB Urimed® Vision

Silicone Self-Adhering Male External Catheter.

## DESCRIPTION/INDICATION

The Urimed® Vision Male External Catheter is an all silicone self-adhering male external catheter designed for the management of male urinary incontinence. It is available in five sizes ranging from Small (25 mm) to Extra Large (41 mm).

## CONTRAINDICATION

Do not use on irritated or compromised skin.

## PRECAUTION

Do not use if allergic reaction or sensitivity occurs.

## DIRECTIONS : TO APPLY

- 1) Wash penis with mild soap and warm water. Dry thoroughly.
- 2) Trim pubic hair if necessary.
- 3) Open package at perforation.
- 4) To remove plastic insert, squeeze catheter at the top of the white cone and pull to release.

"Ultra" and "Short" versions only

- 5) If uncircumcised, do NOT pull back foreskin.
- 6) Do not apply powders or creams as this may prevent a secure seal.

- 7) Unroll self-adhering catheter over penis.
- 8) Gently squeeze the catheter to properly seal adhesive to the skin.

**IMPORTANT :** Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

- 9) Connect to drainage device.

## DIRECTIONS: TO REMOVE

Gently roll catheter off the penis. Wash penis with mild soap and warm water. Dry thoroughly. Single use.

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- DE B. Braun Melsungen AG-OPM
- GB B. Braun Medical Ltd
- IT B. Braun Milano S.p.A
- CH B. Braun Medical AG
- FI B. Braun Medical Oy
Under license from Rochester Medical Corporation